


## ASSIST SILVER ABSORBENT DRESSING



## DRESSING ARCHITECTURE & INDICATIONS FOR USE

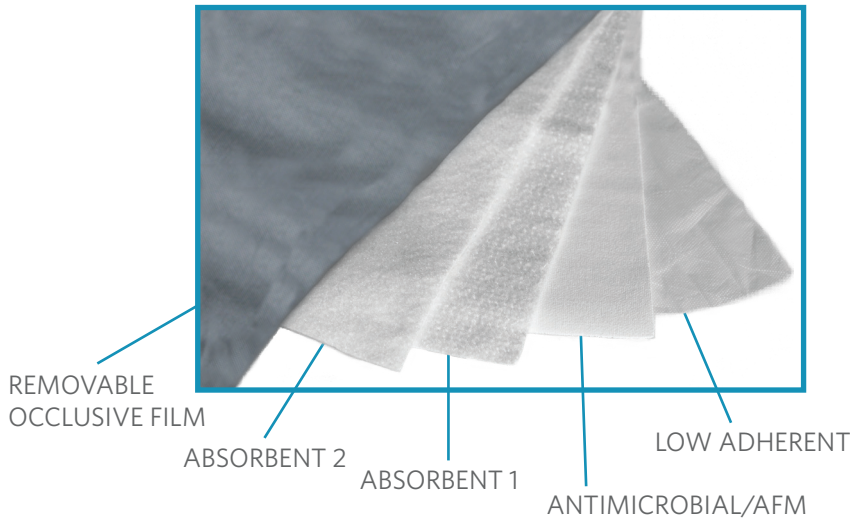

ASSIST Silver Absorbent is indicated for the management of acute and chronic wounds:

- Partial thickness burns
- Incisions
- Skin grafts
- Donor sites
- Lacerations
- Abrasions
- Stage I-IV dermal ulcers: vascular, venous, pressure, and diabetic
- Lymphedema

## BENEFITS OF ASSIST SILVER ABSORBENT DRESSING

- Eliminates need for non-adherent layer
- Saves O.R. time during application
- Effective sustained release of ionic silver for up to seven days<sup>1</sup>
- Minimal maintenance reduces nursing time
- Active Fluid Management® (AFM®) technology is designed to pull and trap excess exudate
- Each dressing can hold up to 1,000cc of fluid at saturation<sup>1</sup>
- May be cut to shape or size
- Grey film can be removed to allow for air flow
- Cost effective


<sup>1</sup>Data on file

# INSTRUCTIONS FOR USE

- Place with the low-adherent layer directly against the wound (**grey side away from the patient**)
- Affix with elastic netting, abdominal belt, roll gauze, stapling to itself, or other common securement methods
- Change the dressing when saturated or soiled
- Remove grey layer if air flow is desired

# APPLICATION EXAMPLES

## DONOR SITES

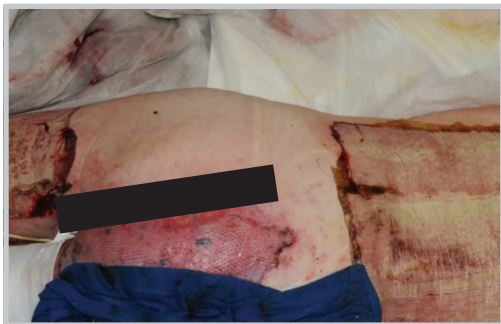

## SJS/TEN ULCERS



## PRODUCT INFORMATION

ASSIST Silver Graft Cover Dressings			
Product Code	Size	Ea/Bx	Ea/Cs
3000051176	8" x 20"	10	30
3000051175	4" x 48"	N/A	10
3000084768	6" x 6"	10	100
ASSIST Silver Absorbent Dressings			
3000117007	Vest	N/A	10
3000116362	18" x 24"	N/A	40
3000006964	24" x 36"	N/A	20
ASSIST Absorbent Dressings			
3000004622	24" x 36"	N/A	25

