COHESIVE BANDAGES

Instructions For Use

- | | | | |
|-----|----------------------------------|-------|--------------------------------------|
| ENG | Flexible Cohesive Bandage | MS | Pembalut jeleket fleksibel |
| AR | ضمادة لاصقة مرنة | NL | Flexibel Cohesief verband |
| CS | Elastická kohezní bandáž | NO | Fleksibel sammenhengende bandasje |
| DA | Fleksibel Sammenhængende Bandage | PL | Bandaż elastyczny przylepny |
| DE | Elastische kohäsive bandage | PT | Ligadura Coesiva Flexível |
| ES | Venda cohesiva flexible | RU | Эластичная самофиксирующаяся повязка |
| FI | Joustava, itseensä tarttuva side | SK | Flexibilný kohezívny obvaz |
| FR | Bandage de contention cohésif | SL | Raztegljivi kohezivni povoji |
| HE | תחבושת אלסטית נצמדת | SV | Flexibelt självhäftande bandage |
| HR | Fleksibilni kohezivni zavoj | ZH-CN | 柔软粘性绷带 |
| IT | Benda coesiva flessibile | ZH-TW | 柔性粘性繃帶 |

Andover Healthcare, Inc.
9 Fanaras Drive, Salisbury, MA 01952 USA
www.ovikhealth.com - 800.432.6686

*Hand-Tear applicable for CoFlex Med, CoFlex NL and CoFlex LF.
Scissors required for CoFlex.

© 2024 OVIK Health, LLC All rights reserved.

Document # COHESIVESIFU2 2024-10
5000018020

CE Advena, Ltd.
Tower Business Centre, 2nd Flr.,
Tower St, Swatar, BKR 4013
Malta

CH REF
Actirep GmbH
Plattnas 3
9476 Weite

Description

These instructions are for CoFlex, CoFlex Med, CoFlex NL and CoFlex LF².

CoFlex cohesive bandage is a cohesive elastic wrap constructed from a nonwoven material and elastic fibers. The cohesive properties allow the wrap to stick to itself but not to other materials or skin. The product may contain natural rubber latex. It is supplied non-sterile.

Indications for Use

CoFlex cohesive bandage is intended for use as an elastic wrap to compress, support, or secure primary dressings or devices.

Contraindications

None known




Precautions & Warnings

- CoFlex self-adherent wrap will not slip or loosen after application. DO NOT wrap too tightly to avoid impairing circulation and potentially causing serious injury.
- Monitor area frequently (no less than every 24 hours) for signs of swelling, discoloration, pain, numbness, tingling, or other changes in sensation. If these symptoms occur, remove wrap immediately and contact your Health Care provider.
- DO NOT place CoFlex cohesive bandages in direct contact with an open wound because it may lead to infection.
- When used as a component in a compression wrap system, it is important to ensure adequate arterial blood flow to avoid injury secondary to decreased circulation.
- CoFlex cohesive bandage should be used as a component of a compression wrap system only under the supervision of a wound care specialist.
- Do Not Reuse. CoFlex cohesive bandage is a single use product.


Directions for Use

General Instructions:

- (1) The CoFlex family of bandages will not loosen or tighten over time, therefore you should wrap the bandage at desired tension and compression.
- (2) Upon completion of wrapping, grasp bandage between fingers and remove stretch - hand-tear or cut the bandage to desired length. Relax the last 2" of bandage and then press down.
- (3) Seal the end of the bandage wrap with your finger, lightly meshing the top of the bandage into the bottom layer. This will provide long staying power.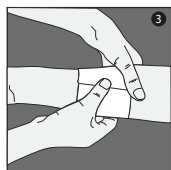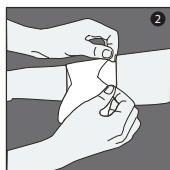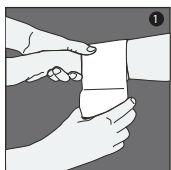

Applying CoFlex Products

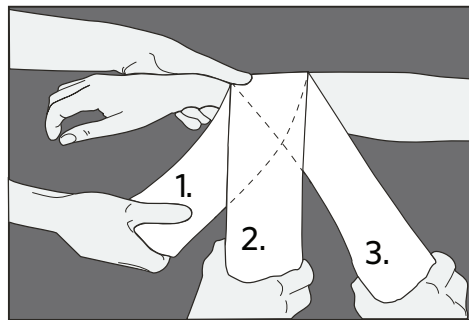1. No to low tension
2. Medium tension
3. High tension

Depending on the specific application for which it is being used, different compression levels of the bandage will be needed. You control the desired tension level. Once wrapped, the bandage will not loosen or tighten and will remain secure.

Removal/Disposal

- CoFlex bandages may be removed by unwrapping or with bandage scissors.
- Dispose of product per your facilities protocol.

Storage

For best results, store CoFlex bandages in original packaging at room temperature until use. Avoid excessive heat and humidity. Refer to the label on each box for expiration date.

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